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REF 17.007.01 - 17.008.90TI

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REF 17.007.01 - 17.008.90TI


Dear valued customer!

We thank you very much for having chosen a high-quality instrument from Helmut Zepf Medizintechnik GmbH.

In order to enjoy this instrument for a long time, we kindly ask you to follow our application notes.

Explanation of symbols	
	Medical device
	Catalog number
	Caution
	CE marking

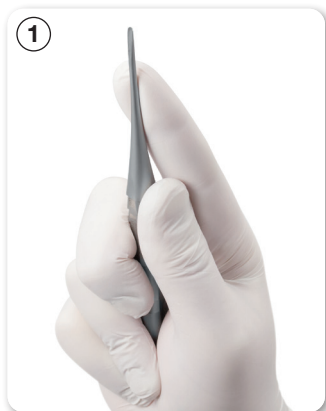MADE IN GERMANY

For the best control, hold the elevator as illustrated.

X-Tools are surgical instruments and should be used as such.

Do not use excessive force.

The instrument's tip is inserted into the sulcus towards the root in a most possible axial direction.

The tip is guided deeper along the root through gentle pressure and twisting movement in apical direction.

For optimal placement of the instrument, the tips are worked out more delicately.

Attention

Usage with leverage and non-planar pressure needs to be avoided.

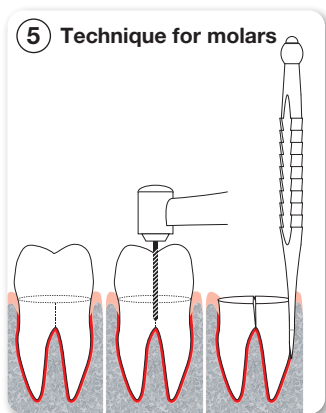
This can cause breakage of the instrument tips.

Attention

We cannot be held responsible for damages caused by improper use.

Note: These X-Tools have been developed as an alternative for classic elevators. The instrument must not be used as a lever.

Rather, the gentle removal of the teeth should be reached by **axial luxation** and cutting the Sharpey's fibers.

Cleaning and Reprocessing

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version.

The form is available for download on our website "<https://www.zepf-dental.com/en/user-information/>".

All medical devices indicated here bear the CE mark.

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